

**ONE-SIZE-FITS-ALL INCISIONS**

**SurgiFish<sup>®</sup>**  
VISCERA RETAINER

**For a faster,  
easier and safer  
abdominal closure.**

- **The first and only...** “one-size-fits-all incisions” viscera retainer specifically designed for closing both large and small incisions.
- **Smooth, pliable and retentive,** SurgiFish<sup>®</sup> provides excellent protection and retraction of the bowel and omentum to the very last inch.
- **Disposable, radiopaque and latex free,** SurgiFish<sup>®</sup> reduces the possibility of retained lap sponges.
- **Cost effective** — with only one size, SurgiFish<sup>®</sup> eliminates the need to stock multiple sizes.



Laparotomy Pack


CE 0336

ISO 9002/EN 46002


**Greer**  
MEDICAL, INC.

# *SurgiFish® is changing the way surgeons are closing the abdomen.*


Used in hospitals throughout the world, SurgiFish® is now included in laparotomy packs and is rapidly replacing lap sponges, malleables and other types of viscera retainers for faster, easier and safer abdominal closures. Our patented “one-size-fits-all” feature allows SurgiFish® to easily adapt to large and small incisions.

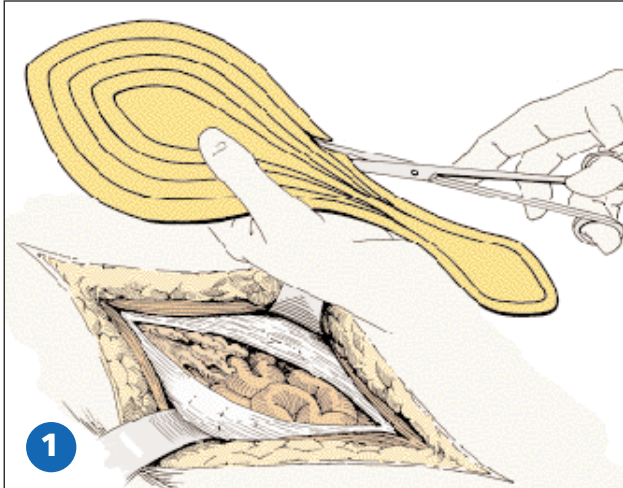


1 Although SurgiFish® need not be cut for most incisions, it is imprinted with contour lines indicating smaller sizes to which the retainer may be cut.

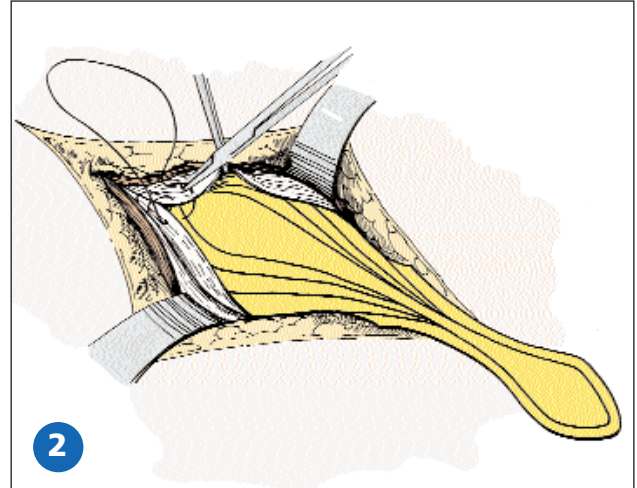

2 Insert SurgiFish® into the incision and begin abdominal closure.

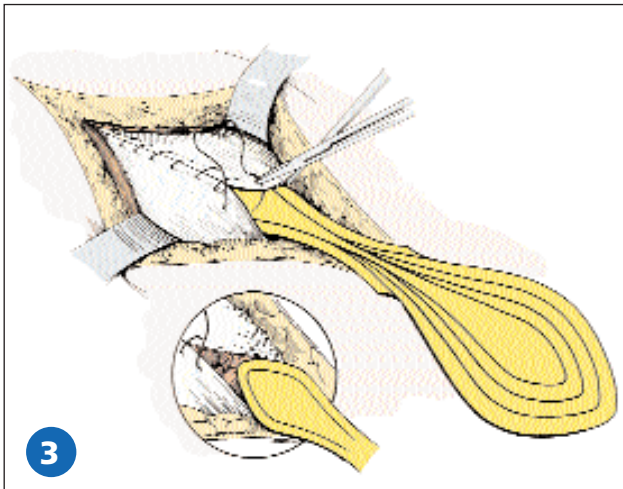

3 When the incision is reduced to a small opening, reverse SurgiFish® and insert the tail portion to complete abdominal closure.

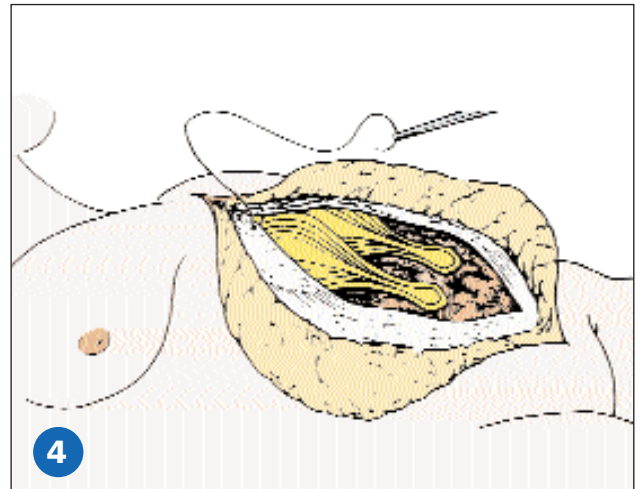

4 For wider and larger abdominal incisions insert two overlapping SurgiFish® for easier abdominal closure.

CATALOG NO.	DESCRIPTION	QUANTITY
4001	Viscera Retainer	1 per package 10 packages per box

**To place your order call 1-800-424-2155**

U.S. Patent No. 4,747,393

## **Greer**

MEDICAL, INC.

314 East Carrillo Street  
Santa Barbara, California 93101 USA  
Telephone: 805-962-5883  
Fax: (805) 962-4615  
E-mail: greer-medical@msn.com

Authorized Representative in EU:  
EMDAR BV  
IJsselburcht 3  
Postbus 5486  
6802 El Arnhem  
The Netherlands